D CARE CON Urinkondom

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GB CARE CON Male External Catheter

DESCRIPTION

Male external catheter is designed for male adult for the management of urinary incontinence. It is 100% silicone and latex-free. Male external catheter consists of sheath, funnel and connector for attaching to a bedside bag or leg bag.

FEATURES

- The Male External Catheter is made from 100% silicone and is very skin-friendly.
- The transparent material allows the condition of the skin to be assessed without removing the sheath.
- The strong adhesive on the inside ensures good adhesion to the penis.
- The adhesive is positioned so as to ensure excellent functionality and enhanced comfort for the patient.

INTENDED USE

Male External Catheter is designed for the management of male urinary incontinence.

INDICATIONS

The Male External Catheter is used for:

- Urinary incontinence after surgical procedures, e.g. on the prostate or bladder.
- Urge incontinence, e.g. after a stroke, Alzheimer's disease, Parkinson's disease, multiple sclerosis.
- Reflex incontinence, e.g. in paraplegics.
- Urinary incontinence following sphincter muscle injury.

CONTRAINDICATIONS

- Inadequate anatomical conditions due to short penile shaft
- Pronounced phimosis (tight foreskin)
- Lack of cognitive ability, dexterity or willingness on the part of the person affected
- Recurrent skin damage on the penis
- Acute skin diseases, e.g. mycosis in the genital area

Extra vigilance is required:

- During and immediately after systemic antibiotic treatment and/or chemotherapy.
- If surgery has been performed on the penis or urethra (tube that carries urine through the penis) in the past

ADVERSE REACTIONS

Skin allergy

DIRECTIONS FOR USE

- Use the Size Charts to select the appropriate size of Male External Catheter.
- Check the model and size on the product label to ensure they are correct. Open the packaging and check if the product is in undamaged.
- Shave pubic hair to prevent it from protruding into the Male External Catheter. It's especially important to shave at the base of the penis, as otherwise hairs may stick to the Male External Catheter, increasing the risk of skin inflammation at the hair follicles ("folliculitis").
- Wash and dry your hands and penis. Then place the Male External Catheter over the tip of the penis (glans), making sure to leave a gap between the tip of the penis and the end of the funnel.
- Carefully roll the Male External Catheter onto the penis and smooth it down.
- Press the Male External Catheter lightly onto the penis.
- Attach the connector of the Male External Catheter to a bedside bag or leg bag. Pull gently on the connector to ensure that the urine bag is firmly connected to the Male External Catheter.
- To remove the Male External Catheter, first detach the urine bag from the connector. Then carefully roll the sheath of the Male External Catheter off the penis.

WARNINGS / PRECAUTIONS

- Please read the Instructions for Use before using the product. Improper use can lead to illnesses (e.g. infections) or injuries to the penis.
- Specialist advice is available from healthcare professionals for professional directions. If you have any questions while using the product, please contact a healthcare professional.
- Do not use if the packaging is damaged or opened.

- Do not use a damaged or misshapen Male External Catheter.
- Do not use after the expiry date.
- For single use only.
- The maximum wearing time is 24 hours. We recommend changing the Male External Catheter every 24 hours.
- The urinary bedside bag or leg bag must be positioned below the level of the bladder.
- When handling the product, your hands and the application area must be clean and dry in order to avoid infections.
- When changing the Male External Catheter, examine the mucous membrane of the urethra and the skin of the penis. If the skin becomes red or shows signs of infection, stop using the Male External Catheter and consult your physician.
- Do not reuse, reprocess. Reuse, reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in injury, illness or death of the patient. Reuse, reprocessing may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another.
- Dispose of product and packaging in accordance with local waste management regulations.
- If you have a complaint about the product, please contact the manufacturer and the competent authorities at the user's place of residence.

STORAGE CONDITIONS

- Store in a dark, cool and dry place.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

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Well Lead Medical Co., Ltd. | C-4 Jinhu Industrial Estate, Hualong | 511434 Panyu, Guangzhou PEOPLE'S REPUBLIC OF CHINA
Shanghai International Holding Corp. GmbH | Eiffestrasse 80 | 20537 Hamburg | Germany
QS Engineering AG | Erlenstrasse 31 | 4106 Thenwil | Switzerland