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## EN

### CARE *FIX* leg bag holders and one-leg pants for fixation of urinary bags

The bag holders and one-leg pants are made of a washable cotton blend fabric with an integrated leg bag pocket. Silicone strips on the upper band ensure secure fixation. Please read all the information in these Instructions for Use carefully before using the straps.

### CARE *FIX* leg bag holder

Fabric leg bag holder for the thigh with integrated pocket and silicone retaining strips on the upper band for secure fixation.

| Size | Circumference | Order number    | PZN      |
|------|---------------|-----------------|----------|
| M    | 54 cm         | 05/02/12/M/GHC  | 09080377 |
| L    | 58 cm         | 05/02/12/L/GHC  | 09080383 |
| XL   | 61 cm         | 05/02/12/XL/GHC | 09080408 |

Material: 64% cotton, 28% polyester, 8% elastane.

Lower leg cuff with integrated leg bag pocket and silicone retaining strips on the upper band for secure fixation.

| Size | Circumference | Order number    | PZN      |
|------|---------------|-----------------|----------|
| S    | 32 cm         | 05/02/13/S/GHC  | 09080414 |
| M    | 35 cm         | 05/02/13/M/GHC  | 09080420 |
| L    | 37 cm         | 05/02/13/L/GHC  | 09080437 |
| XL   | 40 cm         | 05/02/13/XL/GHC | 19160166 |

Material: 64% cotton, 28% polyester, 8% elastane.

### How to use leg bag holder:

- Pull the bag holder over your foot and fix it in the desired position on your thigh or lower leg. The pocket for the urinary bag is located at the side.
- For the lower leg cuff, position the cut-out over your knee to ensure the cuff sits securely.
- Open the side pocket and insert the leg bag after first checking that the drain tap is closed.
- Pass the drain tap through the opening in the bottom of the pocket.
- Please make sure that the sleeve is not too tight.
- After putting on the bag holder, check that the blood circulation in your leg is normal.

### CARE *FIX* Premium one-leg pants

made of cotton blend fabric with silicone retaining strips on the waistbelt for secure fixation.

| Size | Circumference | Order number     | PZN      |
|------|---------------|------------------|----------|
| M    | 85-95 cm      | 05/03/10/M/GHC   | 09080443 |
| L    | 90-100 cm     | 05/03/10/L/GHC   | 09080466 |
| XL   | 98-120 cm     | 05/03/10/XL/GHC  | 09080472 |
| XXL  | 115-130 cm    | 05/03/10/XXL/GHC | 19160172 |

Material: 87% cotton, 8% elastane, 3% polyester, 2% polyamide

### How to use Premium one-leg pants:

- Open the Velcro fastener of the waistbelt and step into the one leg pant. The pocket for the urinary bag is located on the outside.
- Fasten the waistbelt.
- Open the pocket and insert the leg bag after first checking that the drain tap is closed.
- To prevent the tube of the leg bag from kinking, pass the tube's connection cone through the internal loop.
- Close the pocket again.
- To drain urine, open the bottom Velcro closure located at the side and pull the drain tap through the opening.

## Care instructions:

Washable at 60°C. The Velcro strap should be closed before washing.

Do not tumble dry.

Do not iron.

## Storage:

This product should be stored in its original packaging. Do not store near direct or indirect light or heat sources. Keep out of the reach of children.

## Disposal:

The product can be disposed of with normal household waste.

## Precautions:

It is recommended that you empty your urinary bag when half full. If you have any questions concerning use or correct fitting of the product, or if you notice any unexpected discomfort or skin irritation, you should consult your healthcare professional.

## Warnings:

When putting on the bag holder or one-leg pants, make sure your blood circulation is not restricted by fixing it too tightly. Check the condition of your skin regularly, especially when using our products for the first time. If constriction occurs during use, please contact your healthcare professional. For device complaints, please contact your supplier. Please also report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority at the user's place of residence.

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# GHC German Health Care GmbH
MedPath GmbH | Mies-van-der-Rohe-Strasse 8 | 80807 Munich | Germany
CMC Medical Devices GmbH | Bahnhofstrasse 32 | CH-6300 Zug | Switzerland
GHC German Health Care GmbH | Birkklück 15 24999 Wees | Tel.: +49 4631 57200 0 info@ghc-medical.de | www.ghc-medical.de